


SPAN AMERICA PUMPS

APM2 Pump

THERAPY MODE:

1. Float mode will evenly inflate all cylinders.
2. Alternating mode will switch from inflating cylinders 1 and 3 to inflating cylinders 2 and 4 every 5 minutes .
3. There is a toggle switch where the hose meets the mattress to choose between alternating air or lateral rotation. Lateral rotation mode will alternate between inflating cylinders 1 and 2 and cylinders 3 and 4 every 5 minutes.
Lateral rotation should not be used when the head of the bed is above 30 degrees.

Auto Firm should be used to provide maximum inflation for transfers, ADLs and CPR. Use Auto Firm button, it will turn off automatically after 20 minutes.

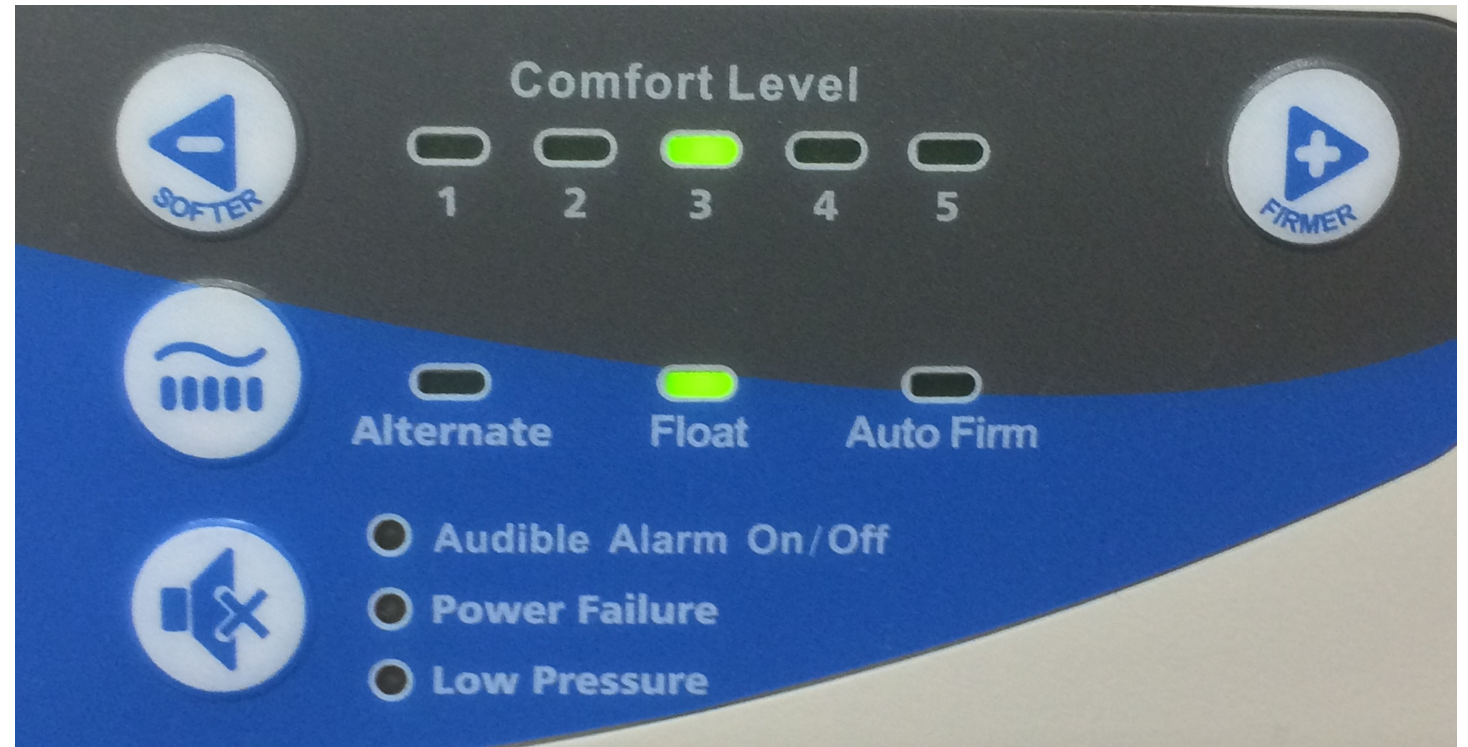

COMFORT LEVEL: Can be adjusted for clients comfort. Clients over 120 lbs should not use level one.

LOW PRESSURE INDICATOR: Will light up upon initial set up and may stay on for up to 30 minutes. If light doesn't go out after 30 minutes-service may be required. If light comes on after initial set up, check that airlines are properly connected and are not kinked.

Audible Alarm: an alarm will sound if either the low pressure or power failure indicator light is on. Press button to silence the alarm.

Power Failure: if power is lost, do not unplug the control unit or disconnect air lines from the control unit.. The air cylinders will equalize pressure in the support cylinders and maintain inflation .

Mattress is READY for USE when: Low pressure light turns off.